PROCEED*

SURGICAL MESH

PROCEED*
KIRURGISK NET

PROCEED*
CHIRURGISCHE MESH
KIRURGINEN

PROCEED* -VERKKO
TREILLIS CHIRURGICAL

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Made in U.S.A.
©ETHICON, INC 2003

DESCRIPTION

PROCEED* Surgical Mesh is a sterile, thin, flexible laminate mesh designed for the repair of hernias and other fascial deficiencies. The mesh product is comprised of an oxidized regenerated cellulose (ORC) fabric, and PROLENE* Soft Mesh, a nonabsorbable polypropylene mesh, which is encapsulated by a polydioxanone polymer. The polypropylene mesh side of the product allows for tissue ingrowth while the ORC side provides a bioresorbable layer that physically separates the polypropylene mesh from underlying tissue and organ surfaces during the wound-healing period to minimize tissue attachment to the mesh. The polydioxanone provides a bond to the ORC layer.

The PROLENE Soft Mesh component is constructed of knitted filaments of extruded polypropylene identical in composition to that used in PROLENE* Polypropylene Suture, Nonabsorbable Surgical Sutures, U.S.P. (ETHICON, Inc.). This material, when used as a suture, has been reported to be non-reactive and to retain its strength indefinitely in clinical use. The PROLENE Soft Mesh affords excellent strength, durability and surgical adaptability, with a porous structure to enable mesh incorporation into surrounding tissues.

The ORC component is an absorbable off-white knitted fabric prepared by the controlled oxidation of regenerated cellulose. The ORC layer is absorbed from the site of implantation within four (4) weeks. Absorption rate depends upon several factors including the amount used and implantation site.

The polydioxanone components made from the polyester, poly (p-dioxanone) polymer that is identical to the polymer used in PDS* II (polydioxanone) Suture, Synthetic Absorbable Suture, U.S.P. (ETHICON, Inc.). The polydioxanone polymer has been found to be nonantigenic, nonpyrogenic and to elicit only a mild tissue reaction during absorption. The polydioxanone component is absorbed within six (6) months.

ACTIONS/PERFORMANCE

PROCEED Mesh is a laminate mesh, whose PROLENE Soft Mesh component is knitted with nonabsorbable fibers, used to reinforce or bridge traumatic or surgical wounds to provide extended support during and following wound healing. The ORC is intended to physically separate the mesh from underlying tissue and organ surfaces during the critical wound healing period, thereby reducing the severity and extent of tissue attachment to the mesh.

Animal studies show that implantation of PROCEED Mesh elicits a transient inflammatory reaction that does not interfere with integration of the mesh into

adjacent tissue. The mesh remains soft and pliable, and normal wound healing is not noticeably impaired.

The ORC and polydioxanone components are essentially absorbed within six (6) months, whereas the polypropylene material is not absorbed, nor is it subject to degradation or weakening by the action of tissue enzymes.

Minimal visceral tissue attachment has been demonstrated in animal studies that show reduction in the extent and severity of adhesions to the mesh.

PROCEED Mesh also does not potentiate infections as shown in animal studies.

INDICATIONS

PROCEED Mesh may be used for the repair of hernias and other fascial deficiencies that require the addition of a reinforcing or bridging material to obtain the desired surgical result.

CONTRAINDICATIONS

None known.

WARNINGS

When this mesh is used in infants, children, pregnant women, or women planning future pregnancies, the surgeon should be aware that this product will not stretch significantly as the patient grows.

As with all foreign substances, PROCEED Mesh should not be placed in a contaminated surgical site.

The mesh may not be used following planned intra-operative or accidental opening of the gastrointestinal tract. Use in these cases may result in contamination of the mesh, which may lead to infection that may require removal of the mesh.

PROCEED Mesh is provided by ETHICON, Inc. as a sterile product. Do not resterilize. Do not use if packaging is opened or damaged. Discard opened unused products.

Users should be familiar with surgical procedures and techniques involving nonabsorbable meshes before using PROCEED Mesh.

PROCEED Mesh has an ORC component, which must not be used in cases in which appropriate hemostasis has not been established. Tissue attachment to the mesh can result if appropriate hemostasis is not achieved.

Foreign body reactions may occur in some patients.

PRECAUTIONS

The mesh should be shaped in such a way that sufficient overlap of the fascial defect on all sides is achieved, thereby allowing adequate stabilization of each of the fascial borders.

The safety and effectiveness of PROCEED Mesh in combination with soaking solutions other than saline, such as peritoneal instillates, and/or medications, have not been studied.

ADVERSE REACTIONS

Potential adverse reactions are those typically associated with surgically implantable materials, including potentiation of infection, inflammation, adhesion formation, fistula formation, and extrusion.

INSTRUCTIONS FOR USE

Correct surface orientation is critical for the PROCEED Mesh to function as intended. The polypropylene mesh side (side with blue stripes) of the product should be placed adjacent to those tissues where tissue ingrowth is desired. The other surface, the ORC side, should be placed adjacent to those tissues where minimal tissue attachment is desired (i.e. visceral surfaces). The orientation is essential. In the presence of a peritoneal defect, the ORC layer should be the side with the potential to contact the viscera.

Appropriate hemostasis should be achieved prior to placement of the PROCEED Mesh.

The mesh should be shaped in such a way that sufficient overlap of the fascial defect on all sides is achieved, thereby allowing adequate stabilization of each of the fascial borders.

To avoid dislodging, crinkling or curling of the edges, PROCEED Mesh should be fixed in place with a sufficient number of non-absorbable sutures, tackers, anchors or staples along the borders of the mesh.

It is recommended that the non-absorbable sutures, tackers, anchors or staples be placed 6.5mm to 12.5 mm (1/4" to 1/2") apart at a distance approximately 6.5mm (1/4") from the edge of the mesh.

Tissue adhesive should not be used for fixation.

STORAGE

Store at 25 °C or less. Brief exposures up to 40 °C is acceptable.

HOW SUPPLIED

PROCEED Mesh is available in single packets as sterile, undyed sheets with blue stripes in a variety of sizes.

CAUTION

Federal (U.S.A.) Law restricts this device to sale by or on the order of a physician.

*Trademark

ETHICON, INC.

a Johnson & Johnson company

Somerville, New Jersey 08876-0151

See Instructions For Use

Use By – Year and Month

Do not reuse/resterilize

Batch number

Method of Sterilization —
Cobalt

Manufacturer

25°C

Store at 25 °C or less

CE mark and identification number
of Notified Body. Product conforms
to the essential requirements of the
Medical Device Directive
93/42/EEC.

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ETHICON GmbH
CH: Johnson & Johnson AG